


Objective *S. pneumoniae* Results at Your Fingertips

- Accurate, objective and automated results in **10 minutes**
- Secure, remote instrument management
- **Excellent performance** compared to molecular and bacterial culture methods
- Integrated data management automatically stores test and user history
- **Flexible, dual mode testing** for a variety of laboratory environments
- User and patient ID captured with **onboard barcode scanner**



Urine/CSF Procedure

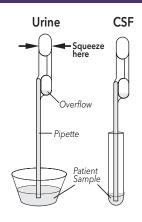




Sofia 2 Development Modes

WALK AWAY Mode – Walk away and multitask

10

READ NOW Mode - Batch multiple samples per hour

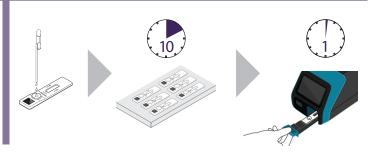

Sofia S. pneumoniae FIA*

- Simple 2-part protocol
- Accurate detection with urine and CSF specimens
- Results in 10 minutes
- Positive and negative liquid controls included
- Room temperature storage
- 24-month shelf life from date of manufacture

Sofia S. pneumoniae FIA Performance Compared to a Commercially Available Qualitative Test - Urine Specimens*

	Compar	ator Test	Positive Agreement	100% (15/15) (95% CI=76%-100%)
	Pos	Neg		
Sofia Pos	15	10**	Negative	95.8% (226/236)
Sofia Neg	0**	226	Agreement	(95% CI=92%-98%)
Total	15	236	Overall Agreement	96% (241/251) (95% CI=93%-98%)

^{**}The 10 specimens that were positive by Sofia S. pneumoniae FIA and negative by the Comparator. Test were concentrated by filtration and tested again on the Comparator Test. All 10 specimens were positive by the Comparator Test. With discrepant result resolution, percent negative agreement = 100% (226/226).

Sofia S. pneumoniae FIA Performance Compared to a Commercially Available Qualitative Test - CSF Specimens*

	oomparator roo		
	Pos	Neg	
Sofia Pos	29	1	
Sofia Neg	0	0	
Total	29	1	

Comparator Test

Positive 100% (29/29) Agreement (95% CI=86%-100%)



^{*}Refer to the Package Insert for additional performance claims.