

# The ART of RSV Testing



Accurate, objective and automated results in as few as 3 minutes

**Excellent performance** compared to molecular and viral culture methods



Secure, remote instrument management through the Virena ecosystem



Integrated data management automatically stores test and user history



Flexible, dual mode testing and simple, CLIA-waived procedures for a variety of laboratory environments



User and patient ID captured with onboard barcode scanner

Sofia 2 with Advance Result Technology Delivering accurate, objective and automated results in as few as three minutes!

8 RSV


\*

Sofia2



Quidel | Rapid Diagnostics | quidel.com | 800.874.1517 | 858.552.1100

#### Sofia RSV FIA – Procedures

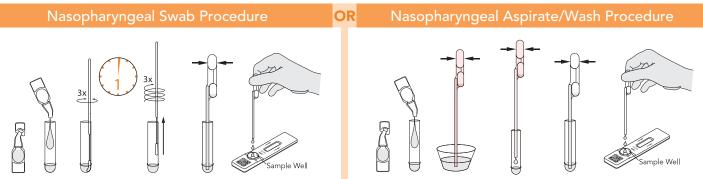

## Sofia 2 Development Modes



### Sofia RSV FIA

- Accurate detection with nasopharyngeal swabs, or aspirate/wash samples
- CLIA waived for children less than 7 years of age
- Moderately complex for pediatric patients 7 to less than 19 years of age
- Excellent performance demonstrated in a multisite study\*
- Room temperature storage
- 24-month shelf life from date of manufacture

#### Sofia RSV FIA Results vs. Culture\*

	Patients ages 0-<19 years		Patients ages 0-<7 years (CLIA Waiver Studies)	
Samples	Sens	Spec	Sens	Spec
NPS	86%	97%	87%	96%
NPA/W	89%	98%	92%	98%
NPS – VTM	87%	97%		
NPA/W – VTM	88%	98%		

\*Refer to the Package Insert for additional performance claims.



