



Sample Collection and Transport Solution:  
***Combo Swab***

Seegene's Solution for

# Safe, Easy and even Reliable Sample Collection

Combo Swab is a user-friendly swab for sampling specimen from nasal and oral cavity.

## Safe & Easy

- No pain through less invasive sampling
- Self-collection under the supervision of healthcare professionals



## Reliable


- Clinician-collected **VS.** Self-collected with Combo Swab
- 96.7% Agreement**  
Indistinguishable to collection by clinician



- Maintaining viability for up to 6 days

## Efficient

- Requires less effort and manpower
- Suitable for mass collection

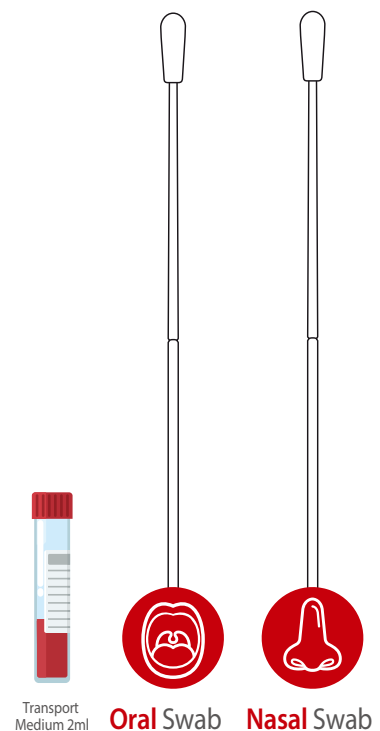

## Sample Collection – Quick Guide

<p><b>01</b></p> <p>Open the pouch. Hold one swab by its tail end.</p>	<p><b>02</b></p> <p><b>Oral</b>      <b>Nasal</b></p> <p>5 times      10 times</p> <p>Rub the area of the mouth 5 times with the first swab, and each nostril 10 times with the second swab.</p>	<p><b>03</b></p> <p>Drop swab into the tube. Locate the cutting point and break off the tail.</p>	<p><b>04</b></p> <p>Close the screw cap tightly.</p>
--	--	---	--